

China (Mainland)

CE RoHS FCC

Skymen

JP-031T

Negotiation

3000pcs per month

In Stock

1pcs / color box, 4pcs / carton

Paypal, , T/T, Western Union, L/C

1

Dental Tool Digital Ultrasonic Cleaner Touch Control Fully Sterilizing **Ultrasonic Denture Cleaner**

Basic Information

- Place of Origin:
- Brand Name:
- Certification:
- Model Number:
- Minimum Order Quantity:
- Price:
- · Packaging Details:
- Delivery Time:
- Payment Terms:
- Supply Ability:



Product Specification

Color:

• Timer:

Silver SUS304 Tank Material: • Housing Material: ABS Capacity: 6.5L • Tank Size: 300×150×150mm 5 Digital Timer Settings 180W • Ultrasonic Power: • Transducer Numer: 3pcs • Application: **Dental Sterilization** • Highlight: stainless steel ultrasonic cleaner,

ultrasonic bath cleaner



More Images



Our Product Introduction

Touch Key Screen Fully Sterilizing Dental Instrument Ultrasonic Cleaner

1.Dental ultrasonic cleaner Is neccessary

Large amount people got infection from hospital,dental office and other medical area each year. So sterilizing process is very important in these place. The first step of sterilizing is cleaning which require fully and high quality cleaning. Ultrasonic cleaner can help you achieve high cleaning level easily.

Comparison between different cleaning method

Spraying	14%
Immersion	30%
Steaming	35%
Brushing	72%
Ultrasonic Cleaning	99%

Dental Ultrasonic Cleaner will help dentist deal with daily cleaning job.

2.Advantage of JP-2500

(1)Oxidation will be prevent and cleaning effect will be improved becase 2500 has degas function.

(2)2 pcs transducer for 2.5L machine is the most powerful ultrasonic cleaner for dental office

(3) If you have different cleaning requirement, just switch your power.

(4)1-30 mins adjustable will meet your large quantity cleaning requirement.

(5)40KHz is the most suitable frequency for dental tool

3. How Skyme Dental ultrasonic cleaner Work For Dental Tool?

(1) Fill water in cleaning tank to meet max line.

(2)Turn on heating function and put some medical cleaning solution before you put dental tool inside.

(3)Put your dental tool in cleaning tank and turn on ultrasonic cleaning button

(4)Set cleaning time you need. It depends how dirty your dental tool is.

(5)Take out your dental tool and drain water if you don't continue to use it.

(6)Dry cleaning tank when you done all cleaning process and disconnect power line.

4.Specification

Model	JP-031T	Frequency	40,000Hz
Tank capacity	6.5L	Timer	1-30mins
Tank size	300×150×150 mm	Unit size	325×180×280 mm
Box packing	400×255×355 mm	Function	Degasing/Power switch
Ultrasonic power	180W	Heating power	200W
N.W.	5.2kg	G.W.	5.6kg
POWER SUPPLY	AC 100 ~ 120V, 60F	Iz ; AC 220 ~ 240V, 50)Hz

Detailed Picture

5.Detailed picture

Breif introduction:



Other Recommendation

6.Other recommendation:


If this tank size is not suitable for your dental tool cleaning, we have other smaller or bigger machine for you to choose.Please check other model below:


7.Introduction of Ultrasonic cleaning

DIRT ELIMINATION PROCESS THROUGH ULTRASONIC TECHNOLOGY

 Cleaning effect will be much better if using the machine together with proper solvent.



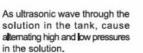

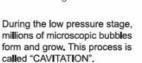

During the high pressure stage, the bubbles implode releasing enormous amounts of energy. They work in all directions, attacking surface and invading all recesses and openings.

8. Skymen Policy For Distributor

We are only specializing in designing and producing ultrasonic cleaner so we need our partners in medical industry, always! We will keep our quality and competitive price for our distributor. If you would like to cooperate with us, please contact me directly.

9. Skymen Advantage

We have been in this industry for 10 years and we are supplying machines to all seller all over the world. All distributor is welcome for win-win coperation.

We have passes ISO, High Tech Company, CE, RoHS, FCC, National Medical Instrument Approval.

ISO:

CE

Competitive Advantage:

1. Our factory Machines produced with good quality parts, and competitive price. Machine warranty is for One year, for all the machines itself quality problems which in normal use and under warranty, we'll provide maintenance and provide machine parts free of charge.

2. For all our machines before delivery had passed Skymen Limited strict inspection, at least be testing 24-48 hours before delivery.

3. Provide ODM and OEM, Customers only need to told us the products which request for cleaning, and their detailed cleaning requirements, our engineers will provide them perfect solution free of charge. Also, clients own machine design and requirements all be OK.

4. After-sale service: To make sure our high quality and top service from us, before order will send the machine design drawing and specifications to clients for confirmation, before machine delivery we'll testing the machines 24-48 hours and QC fully inspection, if there is any problem in Customer's place, we'll helping to checking out the problem and provide the machine parts for replace free of charge, if can't to solve please return to us for a new one replace.

Skymen Technology Corporation Limited			
🕓 86-755-27094405 🔄 info@skymen.cc 🧔 skymenultrasonic.com			
Floor 1st & 2nd, Building 3, Tanggang Taifeng Industrial Park, Dawangshan Community, Shajing Street, Bao'an District, Shenzhen			